

Practice Waste Solutions Amalgam Bucket Recycling & Shipping Instructions

Usage of your PWS Amalgam Bucket:

The PWS Amalgam Bucket should be used for the storage and recycling of 'dry' amalgam waste, such as:

- Contact & non-contact amalgam
- Spent amalgam capsules
- Teeth with amalgam
- Chairside traps
- Vacuum pump filters

IMPORTANT! Remove any liquid from vacuum pump filters before placing them into the amalgam bucket. **NO LIQUID MERCURY OR ANY OTHER METALS. WASTE AMALGAM ONLY** should be placed in this bucket.

Complete Amalgam Waste Recycling Form:

- Fill out Part 1 of the "Profile Form for Recycling of Waste Amalgam"
- Sign and date bottom of the form
- Place completed form into the return shipping box
- Use supplied twist tie to close plastic liner bag and place into 5 gallon amalgam bucket
- Place amalgam bucket into shipping box

Shipping Procedure:

- Please locate "Shipping Code" (see below) prior to packing container.
- Visit www.solmetex.com and click on "Compliance Center" in the menu bar.
- Log into the Compliance Center with your email address and password. If this is your first time logging into the portal, you will need to create a login using your Solmetex Acct # (located on your packing slip) and your zip code. Note: If your product was shipped directly from your dealer, you may not have a Solmetex packing slip. In this event, Please contact Solmetex Customer Service for further assistance.
- Within the portal, locate "Shipping Label" in the upper right hand corner of the portal window.
- Under the product drop-down list, choose "Amalgam Bucket"
- Enter your Shipping Code located on the lid of the amalgam bucket, select "submit".
- Website will generate a UPS label.
- Print the label, place into the packing slip envelope, remove backing from envelope and place on box for UPS pick up or call 1-800-742-5877 to schedule pick up.

NOTE: One UPS label per Solmetex shipping box.



NOTE: If no shipping code is shown on lid, then call 800.216.5505 for recycling info.

<p>50 Bearfoot Road Northborough, MA 01532 Phone: (508) 393-5115 Fax: (508) 393-1795</p>		Picking Slip		Date Jan 1, 2019	Page 1	
		Shipment Number SH0000000000000123456				
Sold To: Your Dealer of Choice 1234 Summer Lane AnyTown, MA 01234 USA			Ship To: 000123456 My Family Dentistry 123 Winterville Ln AnyTown, MN 05432			
"Account Number"						
Reference	PO Number	Customer No.	Salesperson	Ship Via	Ship Date	
ORD000000020001	123456789	12345678		000	Jan 02, 2019	
Picking Sequence	Item Number	Description	Qty. Ordered	UOM	Qty. Shipped	Qty. Backord.
	PWS-AB1	PWS Amalgam Bucket	1.0000	EA	1.0000	0.0000

Recycling Certificates may take up to 30 days from date shipped to the recycler to process.

For more information, visit www.solmetex.com, "Live Chat" Mon – Fri, 8:30am – 5:00pm EST or contact us at 800-216-5505.



Profile Form for Recycling of Waste Amalgam



50 Bearfoot Road
Northborough, MA 01532

SHIPPING/BILLING INFORMATION

DATE RECEIVED:	
WASTE NAME/DESCRIPTION: Spent Dental Amalgam	
GENERATOR:	
SHIPPING ADDRESS:	
GENERATOR CONTACT NAME:	EMAIL ADDRESS:
PHONE:	FAX:
BILLING INFORMATION (COMPANY, STREET, CITY, STATE, ZIP):	
BILLING CONTACT NAME:	EMAIL ADDRESS:
BILLING PHONE:	BILLING FAX:
PROCESS GENERATING WASTE (BE SPECIFIC): Spent Dental Amalgam from Dental Facilities	

DOT/EPA INFORMATION

D.O.T. SHIPPING NAME: Spent Dental Amalgam for Recycling, DOT Non-Regulated (<1 lb Mercury)		
PACKING GROUP: None	HAZARD CLASS: None	UN/NA: None
REPORTABLE QUANTITY:	ERG#:	E.P.A. WASTE CODES (IF APPLICABLE): N/A
PHYSICAL DESCRIPTION OF WASTE: Spent Disinfected Dental Amalgam		

CHEMICAL COMPOSITION - CONSTITUENTS <i>(MUST ACCOUNT FOR 100%)</i>	TOTAL %	TCLP (MG/L)	TCLP (MG/L)
MERCURY (< 1 LB)	Up to 50%		
SILVER, ZINC, OTHER METALS	Up to 50%		
BLEACH SOLUTION	Up to 50%		

CERTIFICATION

I hereby certify that I have personally examined and am familiar with the information submitted in this and all attached documents.

Signature

Print Name/Title

Date

Mercury Recyclers International PO Box 62686 Palm Bay, FL 32906 1-888-626-0694	APPROVAL #: <div style="border: 1px solid black; height: 40px; width: 100%;"></div> (MRI Use Only)
---	--