

Syclone Air/Water Separator Return Procedure

1. Fill out "Generator Section" of waste profile form
2. Sign and date, then place in the return shipping box
3. Before shutting off the vacuum:
 - a. Place 1 ½" black pipe cap over both the inlet and outlet pipe
 - b. Insert red plug into hole underneath the air/water separator
4. Place capped system into clear plastic bag and close with twist tie
5. Flip system upside down and place into 6-gallon bucket
6. Place plastic lid onto bucket and secure tightly
7. Place bucket into the return shipping box
8. Use packaging tape (not included) to seal box for shipment
9. Place prepaid return UPS label on box

