

# Hg5 Air Water Separator Recycling & Shipping Instructions

## Air Water Separator Packing Instructions:

- Remove white PVC piping from the air water separator and dispose of any liquid.
- Place both items into plastic bag lining the 5 gallon return bucket.
- Note: no other material is to be placed in this shipment. Do not include any other metals or liquid mercury.
- The Hg5 Collection Container needs to be packaged separately.
- Place amalgam bucket into shipping box.

## Complete Amalgam Waste Recycling Form:

- Fill out Part 1 of the “Profile Form for Recycling of Waste Amalgam”.
- Sign and date bottom of the form.
- Place completed form into the return shipping box.
- Use supplied twist tie to close plastic liner bag and place into 5 gallon amalgam bucket.
- Place amalgam bucket into shipping box.

## Shipping Procedure:

- Please locate “Shipping Code” (see below) prior to packing container.
- Visit [www.solmetex.com](http://www.solmetex.com) and click on “Compliance Center” in the menu bar.
- Log into the Compliance Center with your email address and password.
- Locate “Print Shipping Label” in the upper right hand corner of the portal screen.
- Under the Product drop-down list, choose Amalgam Bucket”.
- Enter your Shipping Code located on the lid of the amalgam bucket, select submit.
- Website will re-direct to the UPS website. Press “Process Shipment”.
- Print the label, place the label into the packing slip envelope, remove backing from envelope and place on box for UPS pick up or call 1-800-742-5877 to schedule pick up.

**NOTE: One UPS label per Solmetex shipping box.**



**NOTE: If no shipping code is shown on lid, then call 800.216.5505 for recycling info.**

		<b>Picking Slip</b>		<table border="1"> <tr> <td>Date</td> <td>Page</td> </tr> <tr> <td>Jan 1, 2019</td> <td>1</td> </tr> <tr> <td colspan="2"> <b>Shipment Number</b>                      SH0000000000000000123456                 </td> </tr> </table>		Date	Page	Jan 1, 2019	1	<b>Shipment Number</b> SH0000000000000000123456	
Date	Page										
Jan 1, 2019	1										
<b>Shipment Number</b> SH0000000000000000123456											
50 Bearfoot Road Northborough, MA 01532 Phone: (508) 393-5115 Fax: (508) 393-1795		<b>“Account Number”</b> 000123456									
<b>Sold To:</b> Your Dealer of Choice 1234 Summer Lane AnyTown, MA 01234 USA		<b>Ship To:</b> My Family Dentistry 123 Winterville Ln AnyTown, MN 05432									
Reference	PO Number	Customer No.	Salesperson	Ship Via	Ship Date						
ORD0000000020001	123456789	12345678		000	Jan 02, 2019						
Picking Sequence	Item Number	Description	Qty. Ordered	UOM	Qty. Shipped	Qty. Backord.					
	HG5-001	HG5 Amalgam Separator	1.0000	EA	1.0000	0.0000					

Recycling Certificates may take up to 30 days from date shipped to the recycler to process.

For more information, visit [www.solmetex.com](http://www.solmetex.com), “Live Chat”  
Monday – Friday, 8:30am – 5:00pm EST or contact us at 800-216-5505.



# Profile Form for Recycling of Waste Amalgam



50 Bearfoot Road  
Northborough, MA 01532

## SHIPPING/BILLING INFORMATION

DATE RECEIVED:	
WASTE NAME/DESCRIPTION: Spent Dental Amalgam	
GENERATOR:	
SHIPPING ADDRESS:	
GENERATOR CONTACT NAME:	EMAIL ADDRESS:
PHONE:	FAX:
BILLING INFORMATION (COMPANY, STREET, CITY, STATE, ZIP):	
BILLING CONTACT NAME:	EMAIL ADDRESS:
BILLING PHONE:	BILLING FAX:
PROCESS GENERATING WASTE (BE SPECIFIC): Spent Dental Amalgam from Dental Facilities	

## DOT/EPA INFORMATION

D.O.T. SHIPPING NAME: Spent Dental Amalgam for Recycling, DOT Non-Regulated (<1 lb Mercury)		
PACKING GROUP: None	HAZARD CLASS: None	UN/NA: None
REPORTABLE QUANTITY:	ERG#:	E.P.A. WASTE CODES (IF APPLICABLE): N/A
PHYSICAL DESCRIPTION OF WASTE: Spent Disinfected Dental Amalgam		

CHEMICAL COMPOSITION - CONSTITUENTS <i>(MUST ACCOUNT FOR 100%)</i>	TOTAL %	TCLP (MG/L)	TCLP (MG/L)
MERCURY (< 1 LB)	Up to 50%		
SILVER, ZINC, OTHER METALS	Up to 50%		
BLEACH SOLUTION	Up to 50%		

## CERTIFICATION

I hereby certify that I have personally examined and am familiar with the information submitted in this and all attached documents.

\_\_\_\_\_  
Signature

\_\_\_\_\_  
Print Name/Title

\_\_\_\_\_  
Date

Mercury Recyclers International PO Box 62686 Palm Bay, FL 32906 1-888-626-0694	<b>APPROVAL #:</b>   <i>(MRI Use Only)</i>
---	---