

## Hg5 Air Water Separator Instructions, Shipping & Profile

In order to prepare Hg5 air water separator for shipment, the white PVC piping must be removed. After the piping is removed, place the air water separator and the piping into the plastic bag lining the 5 gallon return bucket. (Remove any liquid before placing the air water separator into the bag.) No other material can be included in this shipment, if you have a collection container to return, please contact Solmetex for proper disposal information. DO NOT INCLUDE THE CONTAINER IN THIS SHIPMENT.

**WASTE AMALGAM ONLY – NO OTHER METAL  
NO LIQUID MERCURY**

### Waste Amalgam Return Instructions:

- Fill out “generator section” of waste profile form
- Sign and date the bottom of the form
- Place the completed form into the return shipping box
- Use supplied “twist tie” to close the plastic liner
- Place bucket into return shipping box

### Shipping Labels:

- Visit [www.solmetex.com](http://www.solmetex.com), locate and hover over “Recycling Program” in the menu bar.
- Select and click on “Shipping Label” from drop down menu.
- Enter “Account number” and “zip code”
- Click on “Shipping Label”
- Fill in all necessary information.
- Print the label, place the label into the packing slip envelope, remove backing and place on box for UPS pick up or 1-800-742-5877 to schedule pick up.

**Note: One UPS label per Solmetex shipping box.**



| Picking Slip  |             | Date   | Page         |     |              |               |
|---|-------------|--|--------------|-----|--------------|---------------|
| <br>50 Bearfoot Road<br>Northborough, MA 01532<br>Phone: (508) 393-6116<br>Fax: (508) 393-1795 |             | Jan 01, 2014   | 1            |     |              |               |
| Sold To:<br>Your Dealer of choice<br>1234 Summer Lane<br>Anywhere, MA 12345   |             | Shipment Number<br>SH00000000000000000000                                    |              |     |              |               |
| Reference<br>ORD000000000000  |             | Ship To:<br>00012356<br>Dealer 055<br>789 Winter Villa<br>Anywhere, CA 12345 |              |     |              |               |
| PO Number   | Dealer No.  | Salesperson  | Ship Via     |     |              |               |
| 12345678  | 12345678    |  | 000          |     |              |               |
| Ship Date   |             | Jan 02, 2016   |              |     |              |               |
| Picking Sequence  | Item Number | Description  | Qty. Ordered | UOM | Qty. Shipped | Qty. Backord. |
|   | HG5-002CR   | COLLECTION CONTAINER WITH RECYCLING  | 1.0000       | EA  | 1.0000       | 0.0000        |

Visit [www.solmetex.com](http://www.solmetex.com) for more information and “Live Chat” Monday – Friday 8:30 am – 5:00 pm EST or contact us at 800-216-5505. Recycle Certificates may take up to 30 day to process from date shipped to the recycler.



50 Bearfoot Road  
Northborough, MA  
01532

# Profile for Recycling of Waste Amalgam

(Please type or print in ink)

|   |                 |
|---|-----------------|
| Date Received:  |                 |
| Waste Name/Description: Spent Dental Amalgam                                      |                 |
| Generator:  |                 |
| Shipping Address (Street, City, State, Zip):                                      |                 |
| Generator Contact:  | E-Mail Address: |
| Phone:  | Fax:            |
| Billing Information (Company, Street, City, State, Zip):                          |                 |
| Billing Contact:  | E-Mail Address: |
| Phone:  | Fax:            |
| Process Generating Waste-Be Specific: Spent Dental Amalgam from Dental Facilities |                 |

## DOT/EPA Information

|  |      |                                    |                       |                |
|--|------|------------------------------------|-----------------------|----------------|
| DOT Shipping Name:<br>Spent Dental Amalgam for Recycling, DOT Non-Regulated (<1lb mercury) |      | Packing Group:<br>NONE             | Hazard Class:<br>NONE | UN/NA:<br>NONE |
| Reportable Quantity:   | ERG# | EPA Waste Codes if Applicable: N/A | NONE                  |                |
| Physical Description of Waste: Spent Disinfected Amalgam                                   |      |                                    |                       |                |

| Chemical Composition – Constituents:<br>(Must Account for 100%) | Total %   | TCLP (MG/L) | TCLP (MG/L) |
|---|-----------|-------------|-------------|
| Mercury (<1lb)  | Up to 50% |             |             |
| Silver, Zinc, Other Metals                                      | Up to 50% |             |             |
| Bleach Solution   | Up to 50% |             |             |

## Certification

I hereby certify that I have personally examined and am familiar with the information submitted in this and all attached documents.

\_\_\_\_\_  
Signature

\_\_\_\_\_  
Date

\_\_\_\_\_  
Print Name/Title

|                    |                                 |
|--------------------|---------------------------------|
| <b>Approval #:</b> | Mercury Recyclers International |
|                    | PO BOX 62686 Palm Bay FL.       |
|                    | 32906                           |
|                    | 1-888-626-0694                  |
| (MRI Use Only)     |                                 |