



Before you install your new NXT Hg5 Amalgam Separator

With a neutral pH between 6 and 8, PowerScrub is the only vacuum line cleaner recommended for the NXT Hg5 Amalgam Separator.

Before you disrupt your suction lines with the installation of your new amalgam separator, it is important that the lines are free of built-up debris. The cutting of your suction lines during installation will introduce air into the lines causing any accumulated waste to break free. This disruption can potentially cause your amalgam separator to fill prematurely.

For one month before installation of your NXT Hg5, Solmetex recommends that you thoroughly clean your suction lines using PowerScrub Vacuum Line Cleaner. Please be sure to follow the instructions for first time users for the first two cleanings and then for subsequent cleanings use as directed. This month-long procedure should help reduce the amount of built-up waste that is currently in your lines.

Clean evacuation lines ensure that suction lines are safe and fully functioning. Proper maintenance requires only a small investment of time from the dental staff. When proper cleaning protocols are followed and the right cleaning products are used, suction lines will remain free of debris build-up, the suction flow will be uninterrupted, and patients will receive the best possible care.

 Solmetex™ recommends PowerScrub Vacuum Line Cleaner & Sidekick dispenser for use with all Hg5 series Amalgam Separators.