


Stage 5 Replacement Instructions

Stage 5 (SS-1, SS-3, SS-7, SS-10)

1 When the "Dental Water Quality" alarm beeps and lights up red, unscrew the Stage 5 cartridge.


Four sizes available for the Stage 5 cartridge. Make sure to install the correct size cartridge.

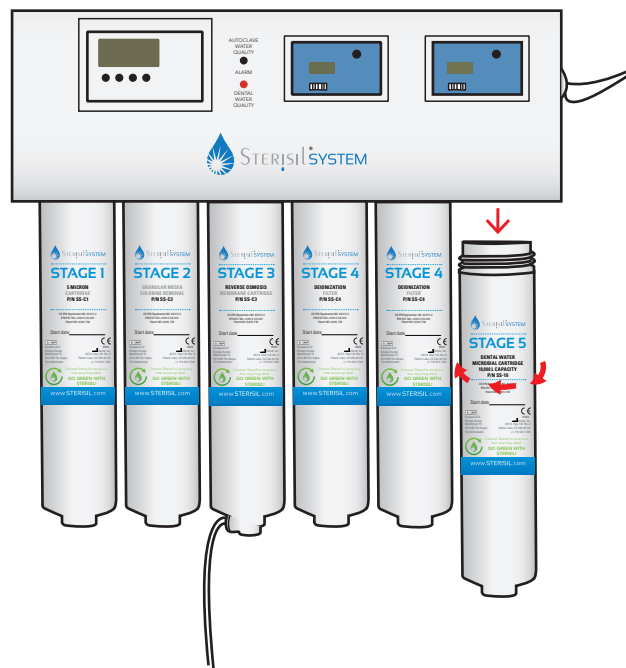

Dispose of cartridge in regular trash disposal



2 Insert appropriate size Stage 5 cartridge into the far right position.

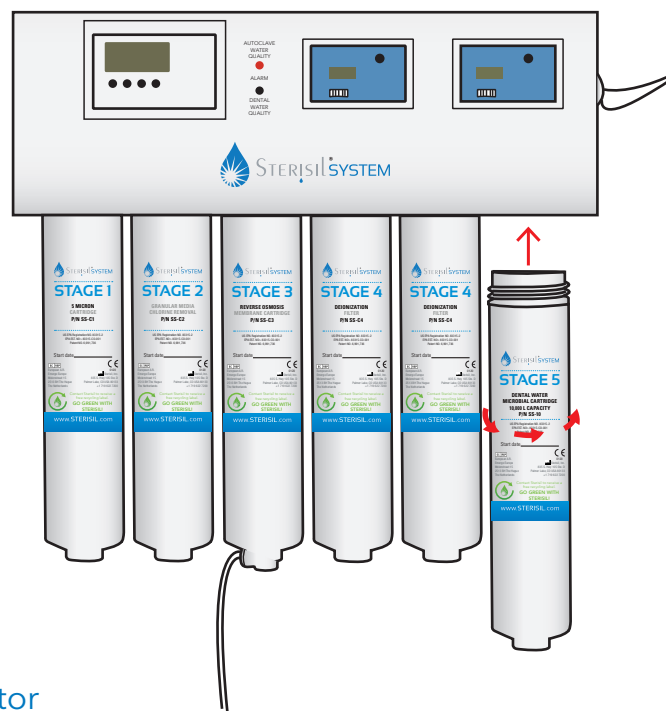

NOTE: After installing the new Stage 5 cartridge, please read the Monitor Reset Instructions to properly reset the Filter Monitor