


How to Install the Sterisil® Straw with the A-dec 500 and 300 Bottles

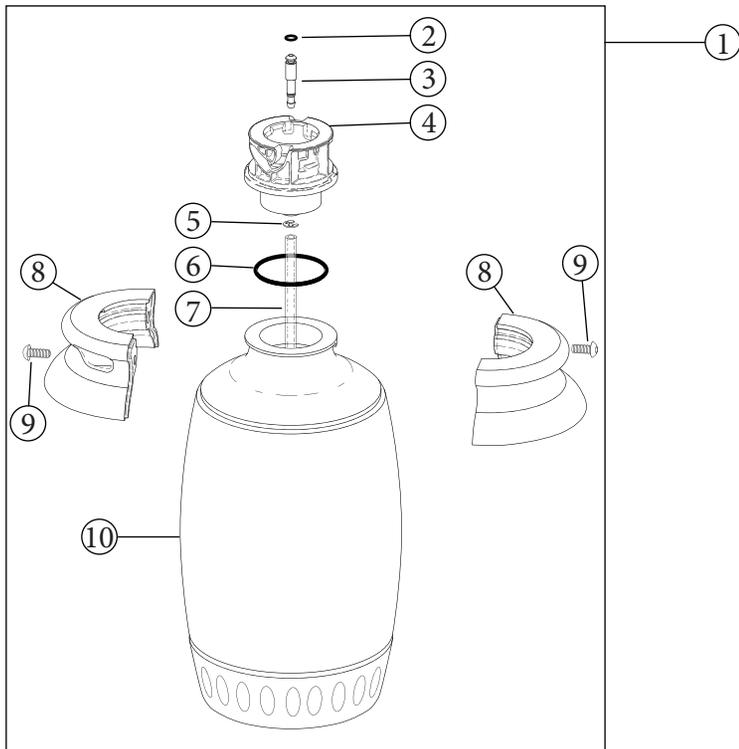

DIRECTIONS

1. Locate the clamp (8) and the two screws (9)
2. Unscrew both screws (9) and pull apart the clamp (8)
3. You now can pull up on the receptacle (4) and access the existing pick up tube (7)
4. Remove existing pick up tube (7) by making a small slit where the tubing connects to the barbed fitting.
5. Attach Sterisil® Straw to the blue 1/4" pick up tube provided
6. Cut the 1/4" pick up tube to the appropriate length so the Sterisil Straw extends about 1/4" from the bottom of the bottle.
7. Slide the Sterisil Straw inside the bottle and reconnect the clamp.

Note: Use care when servicing this water bottle, be sure to align the clamp with the detent on the bottle to successfully reassemble the parts without causing damage.

| REF # | DESCRIPTION |
|-------|--|
| 1 | Water Bottle Assembly |
| 2 | O-ring |
| 3 | Connector, water |
| 4 | Receptacle |
| 5 | Retaining ring |
| 6 | O-ring |
| 7 | Tube, supply, 8" long |
| 8 | Clamp, water bottle |
| 9 | Screw, button head socket, 10-32 x 1/2 |
| 10 | Water bottle with base |

