

Amalgam Instructions, Shipping & Profile

This container is for waste amalgam, amalgam capsules, teeth with amalgam, chairside traps and vacuum pump filters. Please remove any liquid from vacuum pump filters before placing them into the container.

WASTE AMALGAM ONLY – NO OTHER METAL NO LIQUID MERCURY

Waste Amalgam Return Instructions:

- Fill out “generator section” of waste profile form
- Sign and date the bottom of the form
- Place the completed form into the return shipping box
- Use supplied “twist tie” to close the plastic liner
- Place bucket into return shipping box

Shipping Labels:

- Visit www.solmetex.com, locate and hover over “Recycling Program” in the menu bar.
- Select and click on “Shipping Label” from drop down menu.
- Enter “Account number” and “zip code”
- Click on “Shipping Label”
- Fill in all necessary information.
- Print the label, place the label into the packing slip envelope, remove backing and place on box for UPS pick up or 1-800-742-5877 to schedule pick up.

Note: One UPS label per Solmetex shipping box.



Reference		PO Number	Dealer No.	Salesperson	Ship Via	Ship Date
ORD0000000000		12345678	12345678		000	Jan 02, 2016
Picking Sequence	Item Number	Description	Qty. Ordered	UOM	Qty. Shipped	Qty. Backord.
	HGS-000CR	COLLECTION CONTAINER WITH RECYCLING	1.0000	EA	1.0000	0.0000

Visit www.solmetex.com for more information and “Live Chat” Monday – Friday 8:30 am – 5:00 pm EST or contact us at 800-216-5505. Recycle Certificates may take up to 30 day to process from date shipped to the recycler.



50 Bearfoot Road
Northborough, MA
01532

Profile for Recycling of Waste Amalgam

(Please type or print in ink)

Date Received:	
Waste Name/Description: Spent Dental Amalgam	
Generator:	
Shipping Address (Street, City, State, Zip):	
Generator Contact:	E-Mail Address:
Phone:	Fax:
Billing Information (Company, Street, City, State, Zip):	
Billing Contact:	E-Mail Address:
Phone:	Fax:
Process Generating Waste-Be Specific: Spent Dental Amalgam from Dental Facilities	

DOT/EPA Information

DOT Shipping Name: Spent Dental Amalgam for Recycling, DOT Non-Regulated (<1lb mercury)		Packing Group: NONE	Hazard Class: NONE	UN/NA: NONE
Reportable Quantity:	ERG#	EPA Waste Codes if Applicable: N/A	NONE	
Physical Description of Waste: Spent Disinfected Amalgam				

Chemical Composition – Constituents: (Must Account for 100%)	Total %	TCLP (MG/L)	TCLP (MG/L)
Mercury (<1lb)	Up to 50%		
Silver, Zinc, Other Metals	Up to 50%		
Bleach Solution	Up to 50%		

Certification

I hereby certify that I have personally examined and am familiar with the information submitted in this and all attached documents.

Signature

Date

Print Name/Title

Approval #:	Mercury Recyclers International PO BOX 62686 Palm Bay FL. 32906 1-888-626-0694
(MRI Use Only)	