

Change-Out Instructions



Hg5 Amalgam Separator → NXT Hg5 Amalgam Separator



Why do I need this hardware replacement kit?



Good question. Solmetex is now providing the hardware necessary to change-out the old Hg5 Amalgam Separator to the new NXT Hg5 Amalgam Separator without having to remove and replace the blue backplate. With the supplied hardware, you can keep the existing backplate on the wall, and change only the bracket, screws, nuts & bolts to install your new NXT Hg5 Amalgam Separator. It's that easy!

Hg5 Top Mount Bracket

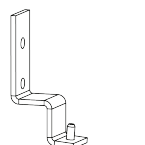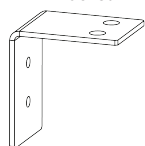

Hg5 Bottom Mount Bracket


NXT RPL Top Mount Bracket

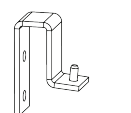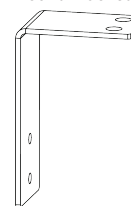

NXT RPL Bottom Mount Bracket

PROCEDURE:

1. Inspect the Hg5 air/water separator to insure there is no liquid waste in it. If liquid waste is present, contact Solmetex for further instructions.
2. If no liquid waste is present in the air/water separator, then with the vacuum running, detach the Hg5 Collection Container from the Hg5 air/water separator by removing the retaining pins and tilting the container to break vacuum seal.
3. Prepare Hg5 Collection Container for recycling by snapping plastic cap onto the Hg5 Collection Container and place into the Hg5 Series Collection Container Shipping Bag and seal according to instructions on the bag. Finally, place bag into Recycle Kit box.
4. Next, turn vacuum off and detach vacuum hosing from the inlet port and outlet port of the air/water separator.
5. Using a 7/16" wrench, loosen both top nuts on the upper bracket.
6. Lift bracket up and remove the air/water separator from the wall backplate and place into the provided plastic bag and seal according to instructions on the bag.

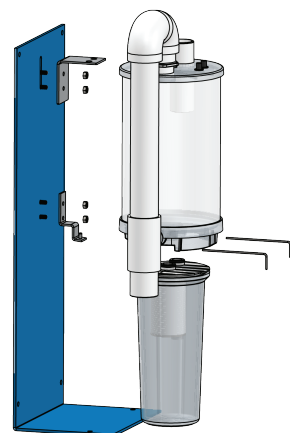

Congratulations on your upgrade to the latest in Amalgam Separation Technology

7. Next, remove the top and lower bracket hardware from the backplate, insuring that the screws remain in place.

8. Take the new NXT adapter hardware and attach the lower bracket first, fastening and securing the nuts.

9. Adjust screws in order to attach the top bracket, place the nuts on, but do not fully tighten.

10. Mount the new NXT Hg5 air/water separator onto the lower bracket.


11. Next, align with the top bracket by inserting the black rubber tabs into bracket holes and tighten all nuts in the upper bracket.

12. Reattach vacuum hoses to the air/water separator.

13. Insert the new NXT Hg5 Collection Container into the air/water separator and secure in place using the retaining pins.

14. Restart the vacuum.

15. Finally, recycle the old Hg5 air/water separator and recycle the Hg5 Collection Container. Package and ship according to the Solmetex recycling instructions.

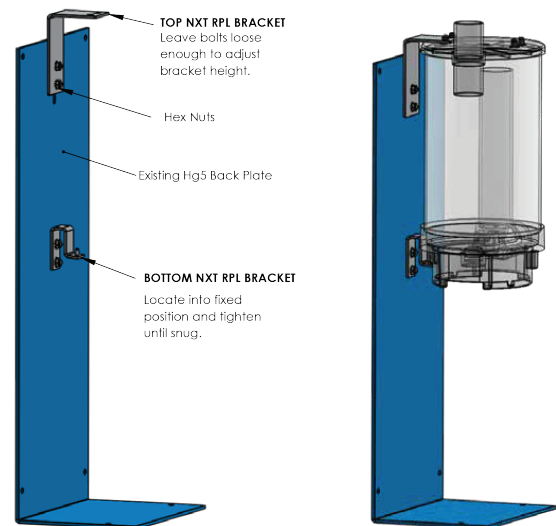

NOTE: The NXT RPL may be installed in either a "right hand" or "left hand" position. Install as best fits your application.

