

Hg5 Air Water Separator Recycling & Shipping Instructions

Air Water Separator Packing Instructions:

- Open and place plastic bag bucket, remove white PVC piping from the air water separator and dispose of any liquid.
- Place both items into plastic bag lining the 5 gallon return bucket.
- Note: no other material is to be placed in this shipment. Do not include any other metals or liquid mercury.
- The Hg5 Collection Container needs to be packaged separately.
- Place amalgam bucket into shipping box.

Complete Amalgam Waste Recycling Form:

- Fill out Part 1 of the "Profile Form for Recycling of Waste Amalgam".
- Sign and date bottom of the form.
- Place completed form into the return shipping box.
- Make note of shipping code on lid of bucket before proceeding to the next step.
- Use supplied twist tie to close plastic liner bag and place into 5 gallon amalgam bucket.
- Place amalgam bucket into shipping box.

How To Locate your Shipping Code:



Shipping Procedure:

- **Locate "Shipping Code" (see below) prior to packing container.**
- Visit Solmetex.com and click the "Recycling Shipping Label" button on the homepage which will take you to My Solutions Center.
- Sign into My Solutions Center with your email address and password.
- Select Product - "6 Gallon or (5 Gallon) disposal for Upper Chamber" based upon the size of bucket you are using.
- Enter your Shipping Code located on the lid of the amalgam bucket, select Submit.
- A new screen will display your label. *Note: you may need to disable pop-up blockers.*
- Print the label, place the label into the packing slip envelope, remove backing from envelope and place on box for UPS pick up or call 1-800-742-5877 to schedule free pick up.



One UPS label per Solmetex shipping box. Labels can only be used once.

Recycling Certificate:

- Visit [My Solution Center](#) to retrieve your Recycling Certificate. Please note: Recycling Certificates may take up to 30 days from date shipped to the recycler to process.



Profile Form for Recycling of Waste Amalgam

Please fill in fields shown in green. Type or print in ink.



Dentistry's Complete
Provider of Water Solutions

30 Bearfoot Road
Northborough, MA 01532

Date Received:	
Waste Name / Description: Spent Dental Amalgam	
Process Generating Waste: Spent Dental Amalgam from Dental Facilities	
Generator Information	
Office or Doctor Name:	
Shipping Address (Street, City, State, Zip):	
Contact Name:	Email Address:
Phone:	Fax (if applicable):

DOT/EPA Information		
D.O.T. Shipping Name: Spent Dental Amalgam for Recycling, DOT Non-Regulated (<1 lb Mercury)		
Packing Group: None	Hazard Class: None	UN/NA: None
Reportable Quantity:	ERG#:	E.P.A. Waste Codes: N/A (If Applicable)
Physical Description of Waste: Spent Disinfected Dental Amalgam		

Chemical Composition - Constituents: (Must Account for 100%)	Total %	TCLP (MG/L)	TCLP (MG/L)
Mercury (< 1 lb)	Up to 50%		
Silver, Zinc, Other Metals	Up to 50%		
Bleach Solution	Up to 50%		

Certification

I hereby certify that I have personally examined and am familiar with the information submitted in this and all attached documents.

Signature

Print Name/Title

Date

Mercury Recyclers International
PO Box 62686
Palm Bay, FL 32906
1-888-626-0694

APPROVAL #:

(MRI Use Only)