

## Dental Water Sampling Instructions



- KIT INCLUDES:**
- Instructions for Use
  - Dehydrated ice packs (see below instructions)
  - Chain of Custody Sheet(s)
  - Sterile vials in clean sealable bag
  - Insulated mailer

**SHIP SAMPLES TO: 862 HWY 105, PALMER LAKE, CO 80133**

**NOTE: COLLECT SAMPLES AT THE BEGINNING OF THE WEEK AND RETURN THEM VIA NEXT DAY AIR TO ENSURE ARRIVAL BY FRIDAY.**

### 1 PREPARING ICE PACKS

Complete this step prior to the water sampling process. Improper use will result in voided tests and no refunds will be given or an additional payment will be required to replace the water collection kit.

1. Massage the ice packs under warm water for at least 1 minute to activate the gel. Fully expanded ice packs should be approximately  $\frac{3}{4}$  of an inch thick.
2. Thoroughly dry the ice packs and place in the freezer for at least 5 hours until frozen. Freeze overnight for best results.

### 3 TAKING WATER SAMPLES

Take samples in provided sterile vials Monday - Thursday only. Make sure to ship the water samples on the same day you collect them.

1. Avoid cross contamination: Use sterile gloves, place vial caps face up, and do not touch outlet tip or interior of vial.
2. Collect samples from each dental waterline handpiece or a multisource sample from each dental unit. To make a multisource sample, combine all water sources from one chair or op into one vial.
3. Run water from each source to be tested for a minimum of 10 seconds before filling sample vial.
4. Collect water in the sample vials and seal tightly. Fill the vial completely to ensure adequate sample volume.
5. Using a black permanent marker, mark each vial with a sample # corresponding to the Chain of Custody sheet.
6. Secure the lid on each vial, wipe clean with a disinfectant wipe and place in clean sealable bag provided.
7. Refrigerate the samples if time between sample collection and shipping has exceeded one hour. DO NOT let the water freeze!

### 2 COMPLETING CHAIN OF CUSTODY

All information is required. This information will be used on the water quality report.

1. Carefully document the room/chair numbers and water source descriptions (e.g. Room 1 Air/Water Syringe).
2. It is recommended that a copy of the Chain of Custody be kept for your records.
3. Place Chain of Custody in provided sealable bag to ensure it stays dry during transit.

### 4 SHIPPING SAMPLES

DO NOT SHIP samples Friday – Sunday or during Holidays (call for more info).

1. Place samples in provided sealable bag and wrap the bag with the vials in the frozen pack(s).
2. Place the ice wrapped vials in the insulated mailer along with the Chain of Custody.
3. Close and seal the insulated mailer, and ship using an Overnight return label. Ship samples to the address indicated at the top of this document.
4. Keep package refrigerated after sample collection if time between collection and shipping has exceeded one hour.
5. Ship samples overnight Monday – Thursday to ensure our lab receives them by Friday.

**WATER TEST RESULTS WILL BE SENT VIA EMAIL WITHIN 14 BUSINESS DAYS AFTER RECEIPT OF SAMPLES.**

**WATER TESTING PERFORMED BY AGENICS LABS, LLC.**

For more information contact your **Sterisil® sales representative or local dealer.**

719 622 7200 or online at [Sterisil.com](http://Sterisil.com)

GP13012.A

