


SYSTEM G4 NOTIFICATIONS

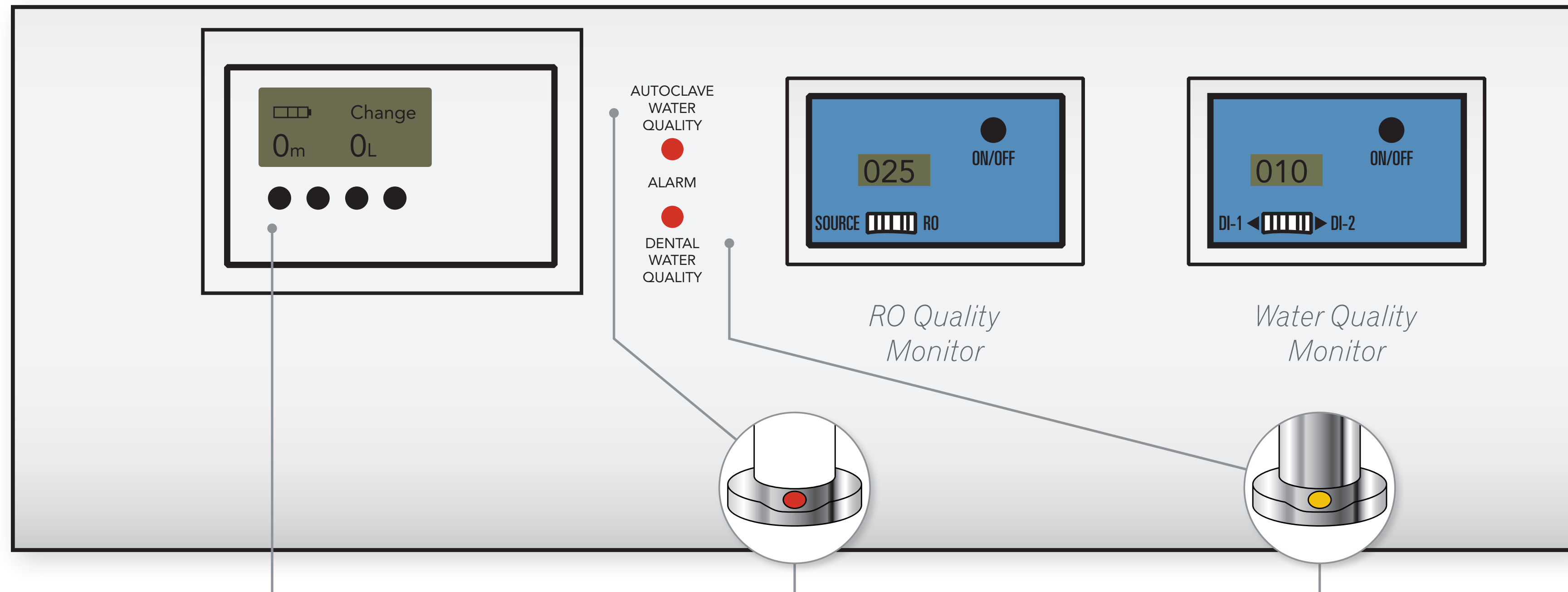

Filter Monitor

MONTH COUNTS (Audible Alarm)

When the count reaches 0m, an intermittent audible alarm will sound, and the word “change” will appear at the top of the Filter Monitor indicating its time to change cartridges 1-3 and the UV light.

LITER COUNTS (Audible Alarm, Stage 5 only)

When a liter count reaches its set point, the word “change” will appear in the top section of the Filter Monitor.

CHANGE BATTERY

The battery icon will appear in the upper left portion of the Filter Monitor, signaling that a battery (AA) change is needed for the Filter Monitor. Be sure to take note of existing month and liter counts prior to changing batteries.

Autoclave Water Quality/
Autoclave Faucet Disc

RED INDICATOR (Visual and Audible Alarm)

The effluent water from the lead (leftmost) Stage 4 Cartridge has reached 10ppm and is due to be changed. DI-1 on the Water Quality Monitor will read at or above 10ppm.

Dental Water Quality/
Dental Faucet Disc

YELLOW INDICATOR (No Audible Alarm)

The Stage 5 Cartridge has processed 90% of its liter capacity.

RED INDICATOR (Visual and Audible Alarm)

The Stage 5 Cartridge has processed 100% of its liter capacity and is due to be changed.