



Instructions for Re-Setting the Filter Monitor: When Replacing Stages 1, 2, 3 & UV light

MUST change Stage 1,2, 3 and UV light and batteries in the filter monitor **ANNUALLY!**
Every time you change Stages 1-3 and the UV Bulb you must reset your monitor to ensure compliance.


1. Turn on the Filter Monitor by pressing the PWR/STG button.
2. Ensure the monitor is highlighting **Stage 1**. If it is not, press the PWR/STG button to cycle through the stages until Stage 1 is selected.

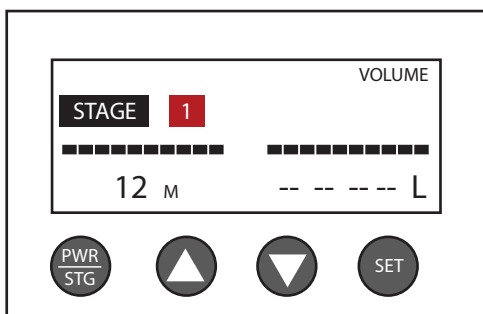


3. Once **Stage 1** is highlighted, press the SET button. The word 'set' should appear near the upper right corner of the screen. Note that the set mode will turn off if no buttons are pressed for 15 seconds. The SET button will then have to be pressed again.

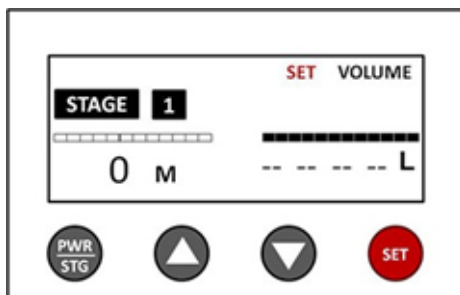


4. Ensure the black line is above month timer (left side). This line can be switched from side to side by pressing the PWR/STG button when the monitor is in set mode.

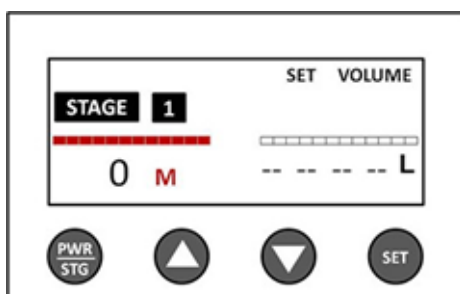


5. Use the arrow buttons to set the value to 12 months.

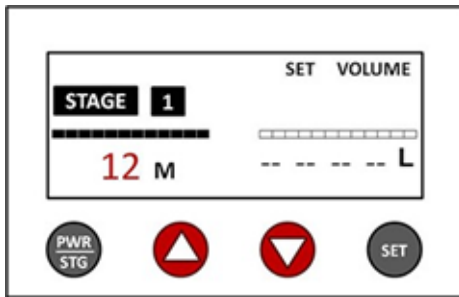


6. Hold down the SET button until you hear 2 beeps. This will save the 12-month setting, and will turn off the set mode.

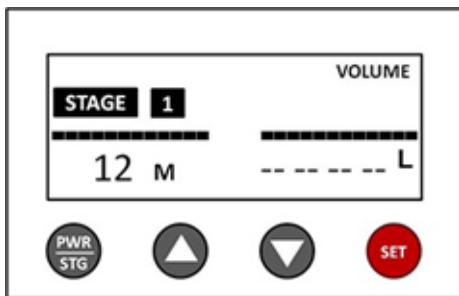

7. Press the PWR/STG button to get to Stage 2. Deactivate the months and liters timers. If the screen looks like the one on the left, skip to Step 16. If there are numbers for months and liters (for example, like the screen on the right), go to Step 8.

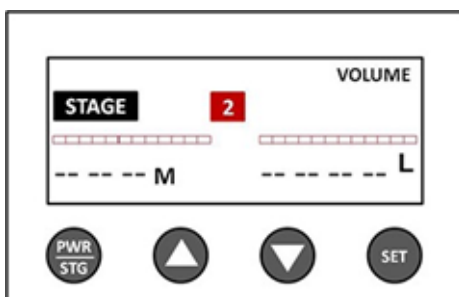


(Go to Step 8)

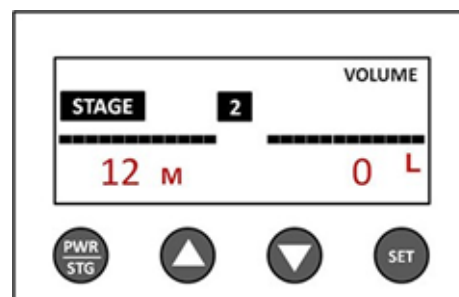


(Go to Step 16)


8. Press the SET button to enter set mode.

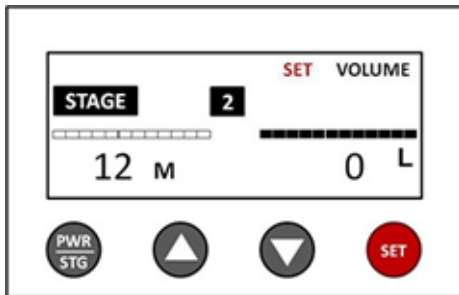


9. Make sure the black bar is over the month value by pressing PWR/STG.

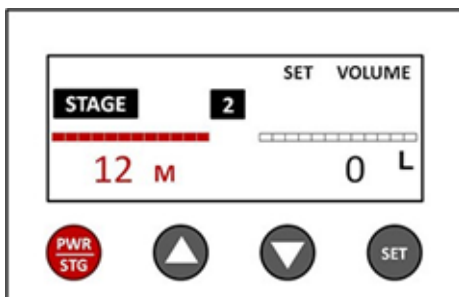


10. Hold the SET and Down Arrow buttons at the same time until you hear a beep. This will deactivate the month timer, showing dashes in the months display.

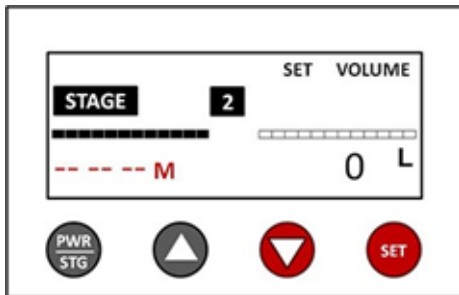


11. Hold down the SET button until you hear 2 beeps. This will save the deactivated month setting.

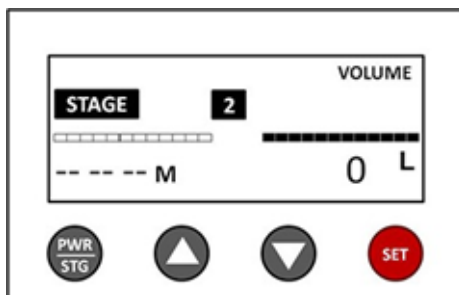


12. If there is a liter value present in this Stage, press the SET button to go back into set mode.

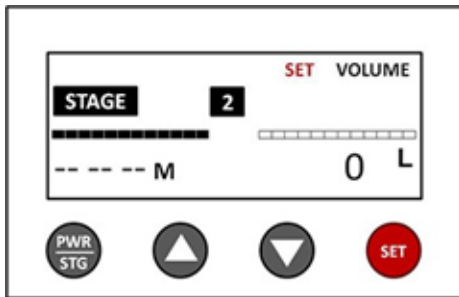


13. Press PWR/STG to get the black bar over the liters display.

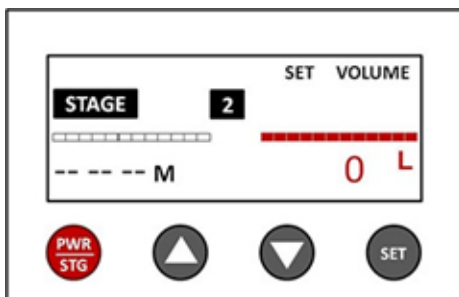


14. Hold the SET and Down Arrow buttons at the same time until you hear a beep. This will deactivate the liter timer, showing dashes in the liter display.

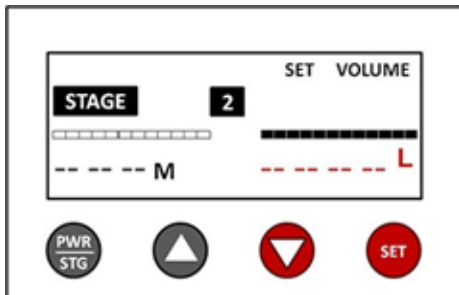


15. Hold down the SET button until you hear 2 beeps. This will save the deactivated liter setting and get out of set mode.

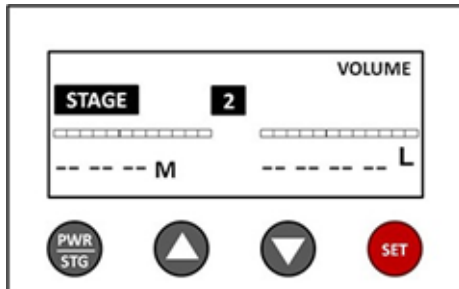

16. Press PWR/STG to get to Stage 3. As with Stage 2, the months and liters timers need to be deactivated. If the screen looks like the one on the left, then you are done. If there are numbers for months and liters (screen on the right), repeat Steps 8-15. The steps are identical, except that it is for Stage 3 instead of Stage 2.

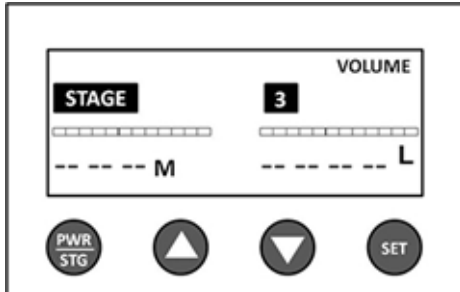

(Reset Complete)

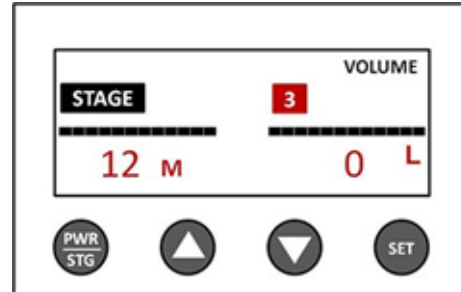

(Go to Step 8)

CONGRATULATIONS!

You have successfully programmed your filter monitor!