



## PROTOCOL FOR SHOCKING WATERLINES

Citrisil™ Shock is the only shock treatment that is compatible with Sterisil's maintenance products. The Sterisil® Straw should remain installed and does not need to be removed before performing a shock procedure.

### HOW TO SHOCK THROUGH THE STERISIL STRAW:

1. Empty water from the bottle, leaving the straw installed
2. Drop a Citrisil tablet to the bottle and add 177 ml (6 fl oz) of warm distilled water or high quality municipal water, depending on your Sterisil straw type
3. Wait 60 seconds for the tablet to fully dissolve
4. Swirl the orange shock solution to coat the inside of the bottle
5. Connect the bottle, with the straw installed, to the dental unit and run lines until the orange color appears in the effluent handpiece water
6. Repeat this process until all handpiece lines are primed with the orange shock solution
7. Leave the lines primed with shock solution overnight
8. The following day empty orange shock solution from bottle and resume operations

**This procedure can ONLY be done with the Sterisil Straw. If you are using an iodine Straw, DO NOT follow the steps above.**

