



STERISIL®

Safe, Simple, Sterisil



Sterisil® Straw V2 Recommended Maintenance Schedule

DAILY

- Leave water bottles on the chairs and water in the lines at the end of the day.

BIWEEKLY

- For ultrasonic scalers and other unused or often stagnant lines, flush for at least 2 minutes.

MONTHLY

- If the chair does not use a Sterisil® Antimicrobial Bottle, shock and clean the bottle per Citrisil™ directions. Leave the Straw on the chair for this procedure. *(If the chair uses a Sterisil Bottle, this procedure is not necessary.)*

YEARLY

- Replace Straw with a new Sterisil Straw V2 and follow shock protocol written in the Straw directions. **Note:** The pick-up tube and luer lock don't need to be replaced.

AS NEEDED

- Remove bottle and top off the water.
- If using model S365-V2: Fill water bottle with distilled water or water that is <100ppm TDS.
- If using model S365m-V2: Fill water bottle with water that is between 100–250ppm TDS.
- If a chair will be dormant for more than two weeks, remove the bottle, purge the waterlines, and let them air dry. Leave the Straw on the pick-up tube with the bottle reattached. Shock with Citrisil shock tablets before using the chair.
- If a chair fails a bacteria test, shock with Citrisil shock tablets. If a chair fails again after shocking, contact Sterisil customer support at 719-622-7200 or support@sterisil.com.

For more information contact your **Sterisil® sales representative or local dealer.**

719 622 7200 or online at Sterisil.com



GP3013.B

