



835 S Highway 105 Suite D
Palmer Lake, CO 80133

Office: 719 622 7200
Fax: 866 299 2495

sterisil.com

IMPORTANT

PROVIDING SAFE WATER FOR YOUR PATIENTS AFTER A 2-WEEK OFFICE CLOSURE

How do you ensure your waterlines are properly treated after closing for 2 weeks?

Prior to closing the dental practice, ensure your office has up-to-date maintenance products in place. For any closures longer than two weeks, remove the bottle, purge the waterlines, and let them air dry.

SHOCK THE WATERLINES WHEN YOU GET BACK INTO THE OFFICE:

Citrisil™ Shock Tablets - Part #: ECS-20 (Box of 20 tablets)

- If using **Citrisil™ Tablets** – Shock with Citrisil Shock tablets before using the chair.
- If using the **Sterisil® Straw** – Leave the Straw on the pick-up tube with the bottle reattached, then shock with Citrisil Shock tablets before using the chair.
- If using the **Sterisil® Cartridge** – Run all handpieces for two minutes every seven days to ensure new residual treatment is introduced into the lines.
- If using a **Sterisil® System (Ac, Ac+, G4 or G5)**
 - a. Bottle Fill: Shock with Citrisil Shock tablets before using the chair.
 - b. Direct Feed: Run all handpieces for two minutes every seven days to ensure new residual treatment is introduced into the lines.

To ensure complete waterline compliance, we recommend testing your water with a third-party lab, such as Agenics Labs. **Call 719-466-5592 or visit agenics.net.**

For questions related to your office situation and to order more Sterisil products, contact your local Sterisil representative.

Please visit ADA.com for the most current closure guidelines in your area.

719 622 7200 or visit Sterisil.com
Safe, Simple, Sterisil