



## Instructions For Use



### 32 oz. Bottle

#### Holding trays and ultrasonic baths:

- Squeeze ½ ounce of solution into dispensing portion of bottle and add to one gallon of tap water. Be sure to mix solution with water in proper ratio.
- Discard used solution at end of the day.

NOTE: Unused solution can be stored for 24 hours.

#### Instrument washers:

- Squeeze 1 ounce of solution into dispensing portion of bottle and add to detergent dispenser with no additional detergent.

**DO NOT** use more than the recommended mix ratio or excess foaming may occur.

### 1 oz. Packets

#### Holding trays and ultrasonic baths:

- Squeeze 1 ounce packet into 2 gallons tap water.
  - Discard used solution at end of day.
- NOTE: Unused solution can be stored for 24 hours.

#### Instrument washers:

- Squeeze 1 ounce packet into detergent dispenser with no additional detergent.

**DO NOT** use more than the recommended mix ratio or excess foaming may occur.

