

IMPORTANT!



Providing Safe Water for Your Patients After a 2-Week Closure

How do you ensure your waterlines are properly treated after closing for 2 weeks?

Prior to closing the dental practice, ensure your office has up-to-date maintenance products in place. For any closures longer than two weeks, remove the bottle, purge the waterlines, and let them air dry.

SHOCK THE WATERLINES WHEN YOU GET BACK INTO THE OFFICE:

Citrisil™ Shock Tablets - Part# ECS-20 (Box of 20 tablets)

If using Citrisil™ Tablets

Shock with Citrisil Shock tablets before using the chair.

If using the Sterisil® Straw

Leave the Straw on the pick-up tube with the bottle reattached, then shock with Citrisil Shock tablets before using the chair.

If using the Sterisil® Cartridge

Run all handpieces for two minutes every seven days to ensure new residual treatment is introduced into the lines.

If using a Sterisil® System (Ac+, G4 or G5)

- Bottle Fill: Shock with Citrisil Shock tablets before using the chair.
- Direct Feed: Run all handpieces for two minutes every seven days to ensure new residual treatment is introduced into the lines.



For questions related to your office situation or to connect with a Sterisil® Water Safety Specialist, contact us at:

SterisilSupport@Solmetex.com | 877-207-1551, Option 1



SAFEWATER
SOLUTION
By Sterisil®

Please visit ADA.com for the most current closure guidelines in your area.

Solmetex.com | 877-207-1551