



SAFEWATER SOLUTION

By Sterisil®

The Sterisil® SAFEWATER Solution Protocol

Follow these simple steps for waterline success.

Ensure consistent waterline safety, compliance, and protection for your patients. Regular in-office testing, combined with proper shock protocols, will help you pass more tests, and keep your operatories up and running.

Best Practice Protocol

Monthly



Test in-office with **FASTCheck15™** Waterline Test Strips



Quarterly



Mail-in with **Sterisil® R2A with FASTRead™** Waterline Lab Test



Annually



Sterisil® Straw with Silver ION Technology

Shock with **Citrisil®**



Most manufacturers and ADS* recommend shocking your dental unit waterlines "regularly."

* Shock after Testing

*ADS- Association for Dental Safety, formerly OSAP

Failed a test?

It happens. Here's what to do:

First do this:
Shock and re-test.

Failed again?
Call us. We can help:
877.207.1551

FASTCheck15™
Instructions
for Use



Frequently Asked Questions



* The Association for Dental Safety (ADS, formerly known as OSAP) suggests testing monthly until passing two consecutive months. Then, transition to quarterly testing as long as you maintain passing results.