

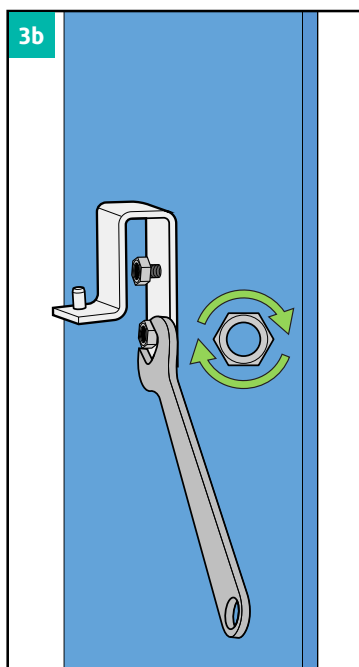
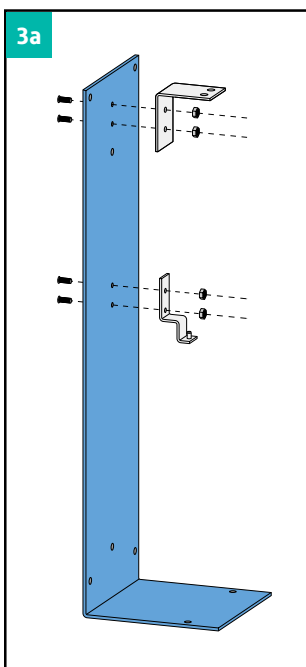


Installing the New NXT Hg5 System on the Existing Backplate

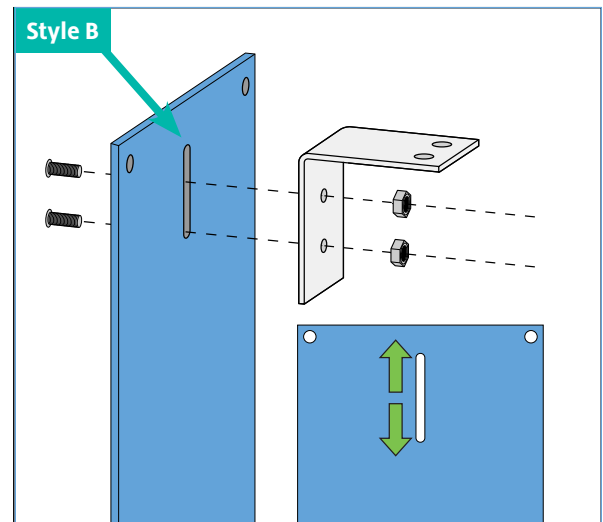



Replacing the mounting hardware

1. Remove old brackets and hardware on the backplate.
2. Remove backplate from wall, if mounted, to allow access to the back side. Install new hardware and brackets (provided) on the backplate
3. Your existing blue backplate will have one of these two backplate styles:
 - **Style A** If your backplate has two holes for the upper bracket, the new brackets should align. If you find that the screw holes do not align with the new upper bracket, contact tech support via chat on [Solmetex.com](https://www.solmetex.com) or at **800-216-5505**.
 - **Style B** If your backplate has one long slot, you simply need to adjust the screw position to put the new bracket on.

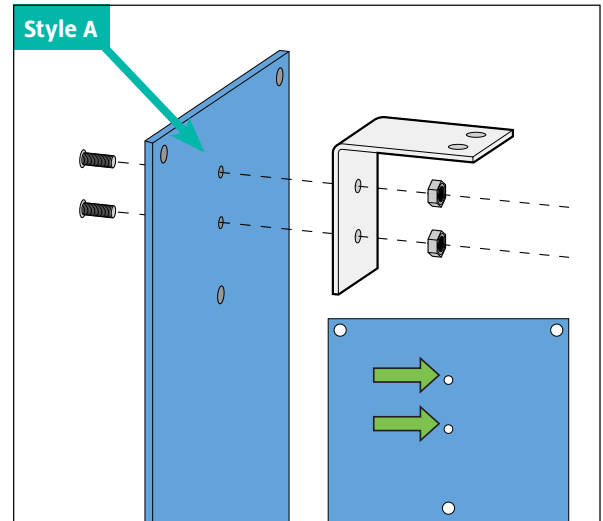


- a. Install 4 threaded inserts from the rear of the backplate and secure using 4 hex nuts on the front of backplate.
 - b. Install the bottom bracket first, tightening hex nut until snug.
 - c. Install top bracket, leaving nuts secure, but slightly loose to allow adjustment when Upper Chamber is installed.
4. Reattach blue backplate with new hardware to wall (if wall mounted).


System Components

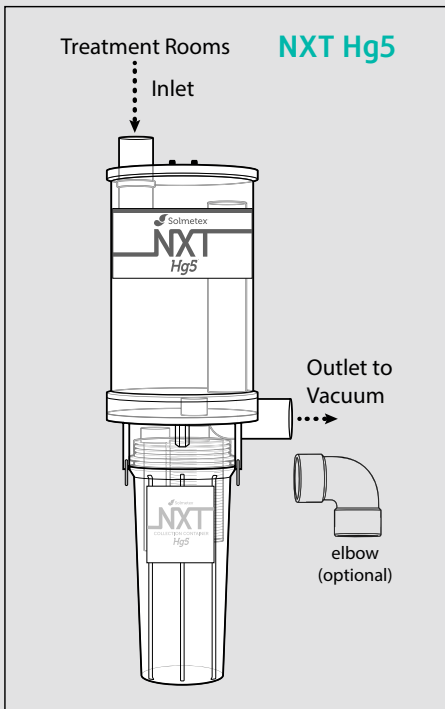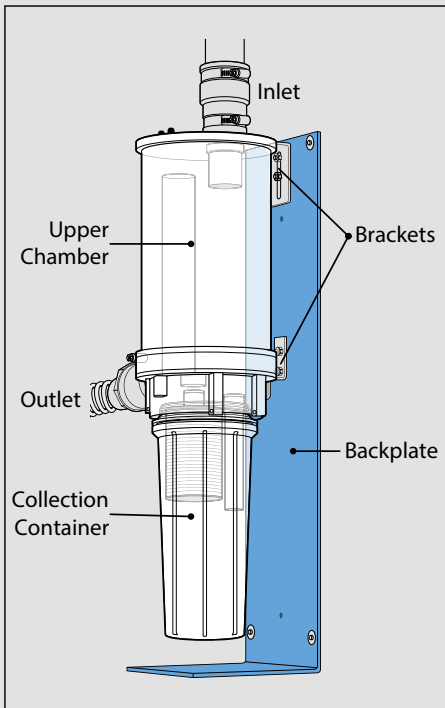

2-3

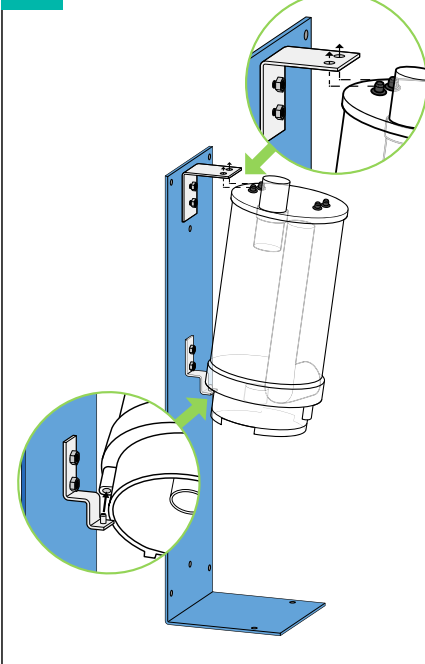



4

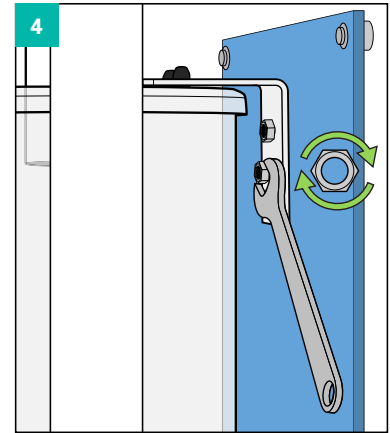

Install the Upper Chamber



1. Confirm if system will be left or right-side oriented based on position of inlet and outlet hoses from prior system.
2. Set and mount the new Upper Chamber onto the lower bracket pin. Rotate assembly into place keeping slot engaged on pin.
3. Align and insert top bracket holes with the rubber tips on top of Upper Chamber, then push the bracket down over tabs until unit is secured in place.
4. Once the Upper Chamber is securely in place, tighten both nuts on the upper bracket.
5. Reattach **(but don't restart) the vacuum**
Reattach the inlet and outlet vacuum line hoses using the rubber fernco couplers. If needed, an elbow is included. Do not glue plumbing to the NXT Hg5 inlet or outlet.
6. Support all connected vacuum tubing and piping to avoid stressing the inlet and outlet connections.

5

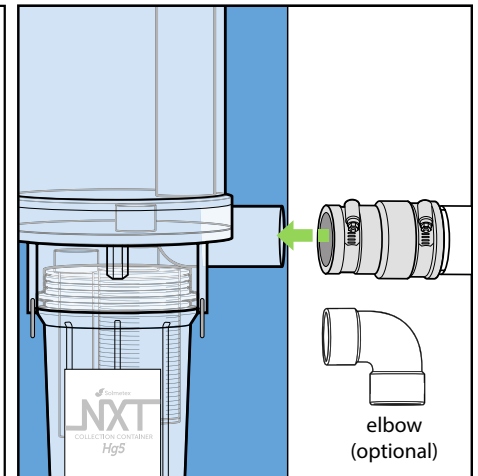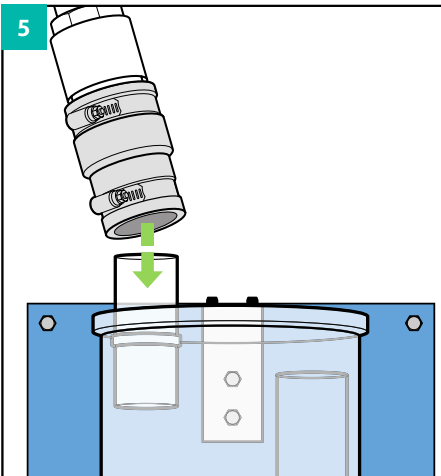

Install the Collection Container

1. Locate the new **NXT Hg5** Collection Container
2. Install your new collection container as follows:
 - a. The collection container can be installed only one way. Note the alignment of the fittings on the collection container with the receiver holes in the upper unit.
 - b. Align the collection container so that the longer nozzle fitting is in line with and on same side as the outlet port on the upper unit.
 - c. Gently insert the collection container fittings into the upper unit receiver holes.
 - d. Continue to push the collection container into the upper unit until it is fully seated.

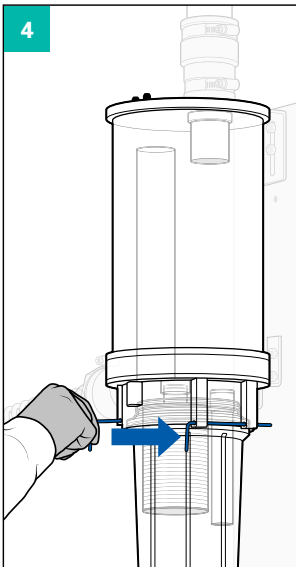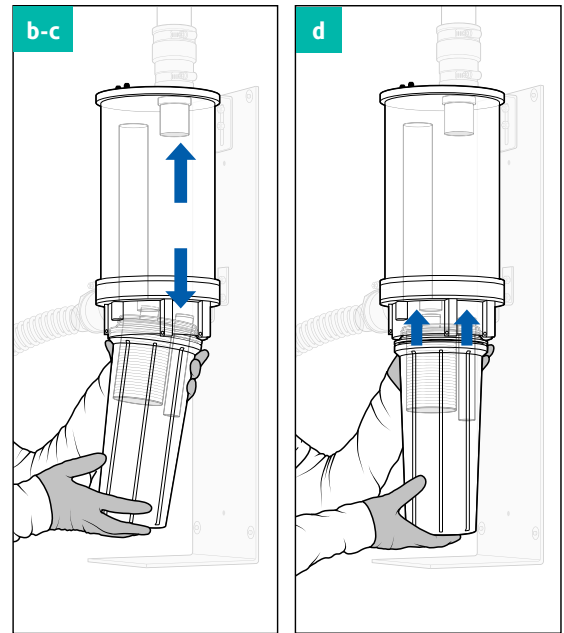

3. Insert two (2) retaining pins into their holes on each side of collection container until seated. The collection container is now locked in position.
4. Fill out the information on label:
 - Write the "date installed" on the new collection container.
 - Write the "Next Change Due" on the new collection container. Choose a date that represents your usual replacement frequency but no more than 12 months in the future.
 - Set a calendar reminder to inspect the collection container.

5. Restart the vacuum

And you're done!



Be sure to ship packages immediately.

Recycling Questions:

Call support at
1.800.216.5505



Packing and Shipping to Recycler

If you haven't already, pack old system items for shipment to the recycler:



Used Hg5[®] Upper Chamber in Bucket



Used Hg5[®] Collection Container in Box